

New



CH-456 CE 0086

- Irregular Heartbeat indicator
- Body movement indicator
- 90 memories function
- Average of last 3 readings
- Easy to read LCD display
- Cuff storage holder
- Automatic power off function
- Simultaneous blood pressure and pulse reading
- Time and date
- AC adapter is optional (AC-230CZ)

Measurement system: Oscillometric method
 Measurement range: Pressure: 0-280 mmHg
 Pulse: 40-180 pulse/min
 Accuracy: Pressure: ±3 mmHg, Pulse: ±5 %
 Power source: AA size x 4
 Optional AC/DC adapter (DC IN: 0.6A)
 Battery life: Manganese battery: Approx. 150 times
 Alkaline battery: Approx. 500 times
 Rated Voltage: DC9V (--- 2W) (--- Direct Current)
 Outside dimensions: W105 x H160 x D122 mm
 Weight: Unit: Approx. 300g (without batteries)
 Cuff: Approx. 130g



CH-456 New

Digital Blood Pressure Monitors

Monitor Your Blood Pressure Regularly. Hyper Tension is a Silent Killer



CE 0086

How to operate



1 Wrap the Cuff around your arm and make sure the blue marker is located in the center of your arm.

2 Place your arm on a table with your palm opened and relaxed. Press the START/STOP switch to start measurement.

3 The blood pressure generally varies in a day. Preferably, you should measure your blood pressure at the same time each day wherever possible.

CH-432 CE 0086

- 90 memories function
- One touch fully automatic
- Average of last 3 readings
- Low Battery consumption
- Silent Inflation

Measurement System: Oscillometric method
 Measurement range: Pressure: 0-280mmHg
 Pulse: 40-180 pulse/min
 Accuracy: Pressure: ± 3 mmHg, Pulse: ± 5 %
 Power Source: AA Size x 4
 Battery life: Manganese battery: Approx. 1000 times
 Alkaline battery: Approx. 1000 times
 Rated Voltage: DC9V (--- 2W) (--- Direct Current)
 Outside dimensions: 129mm(W) x 55mm(H) x 119mm(D)
 Weight: Unit: Approx. 240g (without batteries)
 Cuff: Approx. 130g



Your Area Distributor



Exclusively Imported & Serviced By:
Mehar Healthcare Corporation
 Admin. Off: 16/89, Lane No. 4, Faiz Road, Karol Bagh, New Delhi - 110005
 Phone: 011-25520753, 25538629, 23664115, 23661140
 Fax: 011-25812657, Mobile: 98101-06623
 E-mail: meharhealthcare@rediffmail.com, meharhealthcare@gmail.com
 Website: www.meharhealthcare.com

CH-453 CE 0086

- Irregular Heartbeat indicator
- Body movement indicator
- 90 memories function
- Average of last 3 readings
- Easy to read LCD displays
- Automatic power off function

Measurement System: Oscillometric Method
 Measurement range: Pressure: 0-280 mmHg
 Pulse: 40-180 pulse/min
 Accuracy: Pressure: ± 3 mmHg, Pulse: ± 5 %
 Power Source: AA size x 4
 Battery life: Alkaline battery: Approx. 1000 times
 Rated Voltage: DC9V ---
 Outside dimensions: 130mm(W) x 53mm(H) x 118mm(D)
 Weight: Unit: Approx. 240g (without batteries)
 Cuff: Approx. 100g



CH-437C CE 0086

- 7 memories function
- Easy to read LCD display
- Cuff storage holder
- Automatic power off function
- Simultaneous blood pressure and pulse reading
- AC adapter is optional (AC-230CZ)

Measurement System: Oscillometric method
 Measurement range: Pressure: 0-280 mmHg
 Pulse: 40-180 Pulse/min
 Accuracy: Pressure: ± 3 mmHg, Pulse: ± 5 %
 Power source: AA size x 4
 Optional AC/DC adapter (IV --- 4W)
 Battery life: Manganese battery: Approx. 200 times
 Alkaline battery: Approx. 700 times
 Rated Voltage: DC9V --- 4W (--- Direct Current)
 Outside dimensions: 86mm(W) x 130mm (H) x 118mm(D)
 Weight: Unit: Approx. 300g (without batteries)
 Cuff: Approx. 100g



CH-650 CE 0086

- Irregular Heartbeat Indicator
- Body movement indicator
- 60 memories function
- Average of last 3 readings
- Easy to read LCD display
- Automatic power off function
- With storage case

Measurement System: Oscillometric method
 Measurement range: Pressure: 0-280mmHg
 Pulse: 40-180 pulse/min
 Accuracy: Pressure: ± 3 mmHg, Pulse: ± 5 %
 Power Source: AAA size X2
 Battery life: Alkaline battery: Approx. 400 times
 Rated Voltage: DC9V ---
 Outside dimensions: 68mm(W) x 74mm(H) x 29mm(D)
 Weight: Approx. 100g (without batteries)



CH-452AC CE 0086

- 90 memories function
- Easy to read LCD display
- Average of last 3 readings
- Automatic power off function
- AC adapter is optional (AC-230CZ)

Measurement System: Oscillometric method
 Measurement range: Pressure: 0-280 mmHg
 Pulse: 40-180 Pulse/min
 Accuracy: Pressure: ± 3 mmHg, Pulse: ± 5 %
 Power source: AA size x 4
 Optional AC/DC adapter (IV --- 4W)
 Battery life: Manganese battery: Approx. 1000 times
 Alkaline battery: Approx. 1000 times
 Rated Voltage: DC9V ---
 Outside dimensions: 129mm(W) x 55mm (H) x 119mm(D)
 Weight: Unit: Approx. 240g (without batteries)
 Cuff: Approx. 100g

